



i-STAT 1[®] Analyzer Handheld Software Update

Quick Reference Instructions for Handheld-to-Handheld Transfer

Note: These instructions require at least one updated analyzer

Please refer to “Quick Reference Instructions for Updating the *i-STAT* 1 with Serial Downloaders and Jammlite, using www.pointofcare.abbott” before proceeding

Please refer to *Technical Bulletin: Updating i-STAT 1 Handheld Software Using www.pointofcare.abbott ART: 731335* for detailed instructions




Art: 732160-00B

Rev. Date: 11-Oct-17

i-STAT is a trademark of the Abbott Group of Companies in various jurisdictions

[B.1] Required Materials:

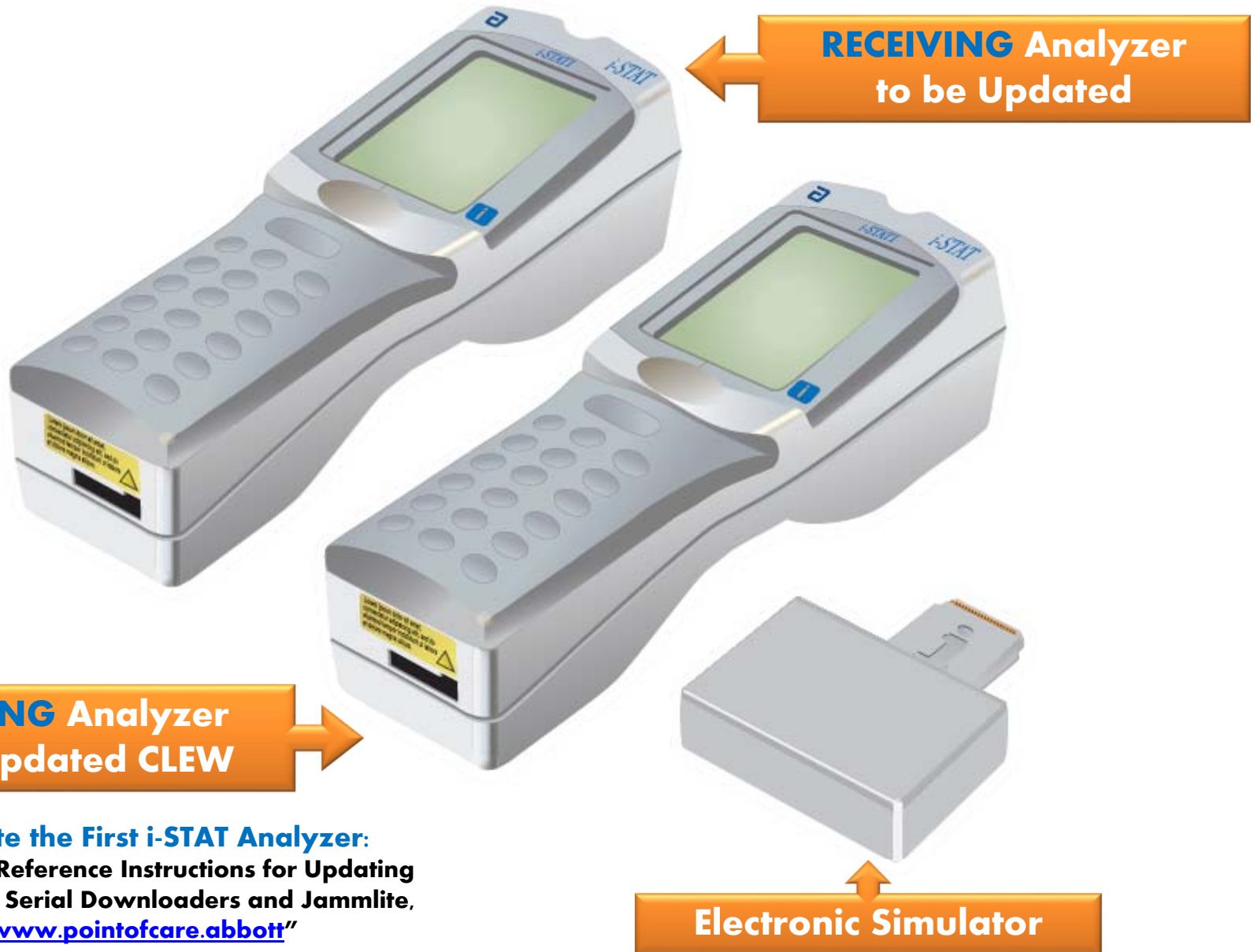


↪ **To Update the First i-STAT Analyzer:**
Refer to “Quick Reference Instructions for Updating the i-STAT 1 with Serial Downloaders and Jammlite, using www.pointofcare.abbott”



[B.2] The **RECEIVING** analyzer should be **OFF**
(Hold the power button until display turns off)

[B.3] Set the **SENDING** analyzer


- Menu Button → Administration Menu
- 7 – Utility
- Enter Password
 - Default Password: Press “Enter” Key
- 1 – Send Software
- 1 – Send

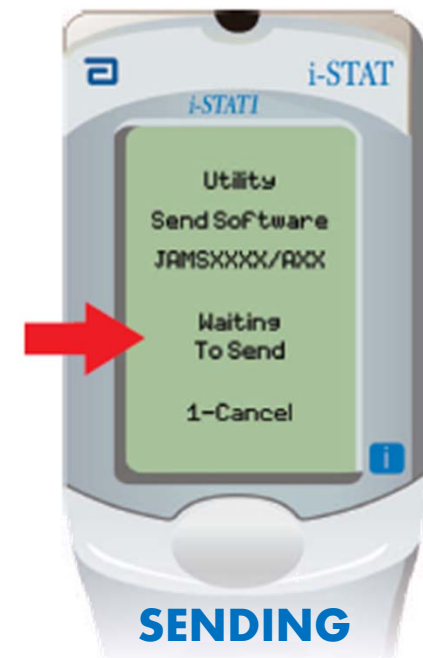

[B.4] Align the infrared windows on both analyzers, bring them closer together




[B.5] The **RECEIVING** analyzer will turn on. Do not move the analyzers

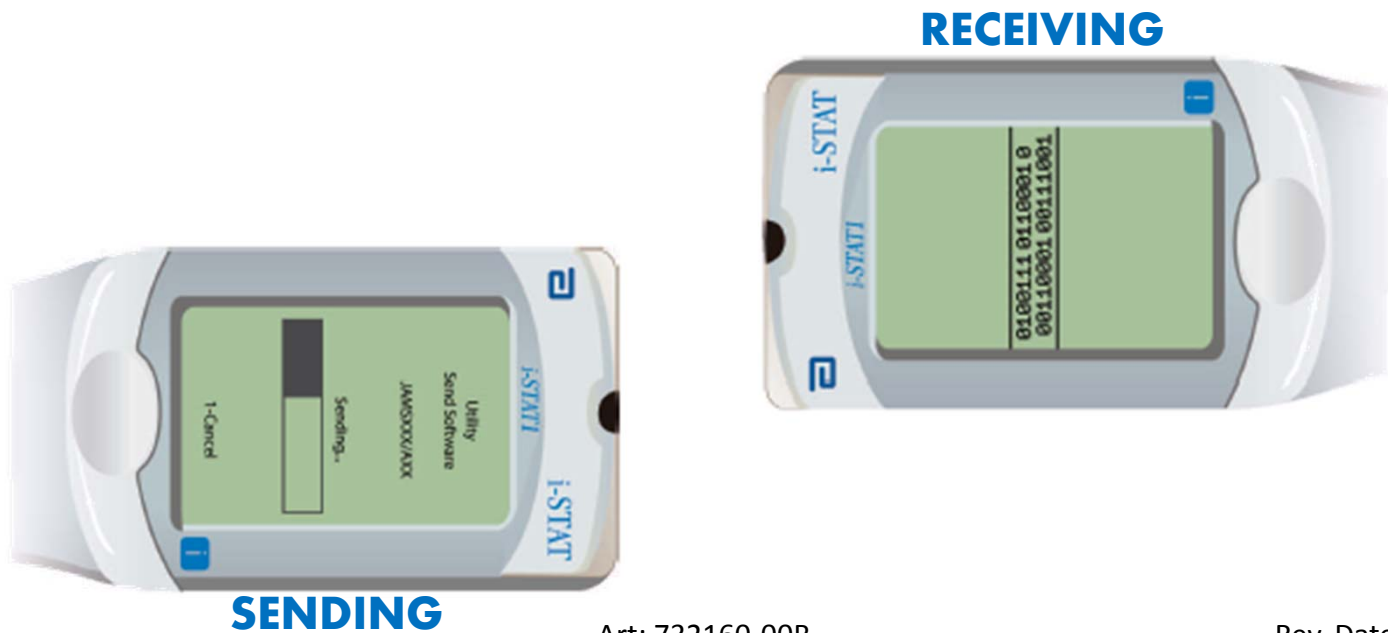

[B.6] Once the installation is complete,
the **SENDING** analyzer will show:

Last Send
JAMSXXXX/AXX
Successful

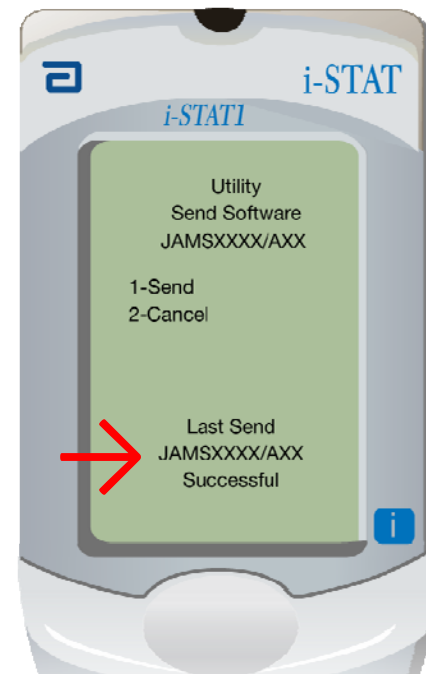

SENDING

**If update is not successful, contact Abbott Point of Care Technical Services:
1-800-366-8020, Option 1, Option 3**

[B.7] Finishing the Software Update Process:

Run the Electronic Simulator on the RECEIVING Analyzer

Analyzer Process Pathway:

- **Power Button**
- **Menu Button → Administration Menu**
- **3 – Quality Tests**
- **4 – Simulator**
- **Follow the on screen instructions**



Update is Complete

To Update Additional i-STAT Analyzers:

- **Handheld-to-Handheld:** Repeat these instructions
- **JAMMLITE:** Refer to “Quick Reference Instructions for Updating the i-STAT 1 with Serial Downloaders and Jammlite, using www.pointofcare.abbott”

To Update Central Data Station:

- Refer to “Quick Reference Instructions for Updating Central Data Station”