



i-STAT 1 TECHNICKÝ PŘEHLED SYSTÉMU

Technická podpora

Společnost Abbott Point of Care a její distributoři jsou odhodláni Vám pomoci při řešení jakýchkoli potíží se systémem i-STAT: s ručním analyzátozem i-STAT 1, kazetami, příslušenstvím i softwarem pro správu údajů. Potřebujete-li technickou podporu ve Spojených státech, volejte prosím na bezplatnou linku technické podpory 800-366-8020. V zemích mimo USA se prosím obraťte na svého místního distributora analyzátorů i-STAT.

SEVERNÍ AMERIKA

USA

Abbott Point of Care
400 College Road East
Princeton, NJ USA 08540
Tel: +800-366-8020, option 1
Tel: +800-284-0702 (waived customers)
Email: techsvc@apoc.abbott.com

KANADA

Abbott Point of Care
400 College Road East
Princeton, NJ USA 08540
Tel: +800-366-8020, option 1
Email: techsvc@apoc.abbott.com

AFRIKA

ANGOLA

Promed International Health Ltd.
Largo AmilcarCabral n 2,A-B-C e 12 R/C
Ingombotas
Luanda, Angola
Tel.No.: +244 94 6146050

BOTSWANA

Leading Edge Markets (Pty) Ltd
P.O. Box 40551
Plot 128 Unit 5
Gaborone International
Finance Park
Gaborone, Botswana

KONŽSKÁ DEMOKRATICKÁ

REPUBLIKA

Wagenia
259 Avenue Wagenia
Kinshasa 02476
Democratic Republic of Congo
Tel.No.: +243 9 97 000097

EGYPT

**International Company for Medical
Equipment S.A.E.**
24th, Gamiet El Dewal El Arabia
Giza, Egypt
Tel.No.: +202 33 47 06 78

KEŇA

Phillips Healthcare Technologies Ltd
Phillips Business Park
Mombasa Road
Nairobi, Kenya
Tel.No.: 254 7336 12025

KEŇA

Ziwala Limited
P.O. Box 17919
Nairobi, Kenya
00100
Tel. No: +254 722 725 529

LESOTHO A MALAWI

Obsidian Health Ltd
Cosmo Business Park
Malibongwe Drive
Randburg, South Africa 2188
Tel.No.: +27 87 3535600

MAYOTTE A OSTROV**RÉUNION**

Abbott France S.A
40/48 rue d'Arcueil
94593 Rungis Cedex, France
Tel.No.: +33 1 4560 2500

NIGÉRIE

JB Consulting (MDP) Ltd
QDT Solution
111 Concorde Road, Building 3,
Level 1 Regus
Maidenhead, Berkshire
SL6 4BY UK
Tel.No.: +44 7808 589 217

SÚDÁN

Penotee Multi Activities Co Ltd.
Building No. 556
Block 22, Badr Street
Altaif, Khartoum Sudan
Tel. No: +249 183 287794

ZAMBIE

Phillips Pharmaceuticals (Zambia)
Plot #7236, Njolwe Road
Light Industrial Area
Lusaka, P.O. Box 36017
Zambia

LIBYE

Al- Haremeen Pharmaceutical
Medical Supplies
Extension of Al Saraj
Street After Audi Showroom
Tripoli, Libya
Tel.No.: 201 00 3333 444

MAROKO

Masterlab Sarl
22 Rue IBN Tayeb Kadiri Les
Orangers Rabat
Morocco
Tel. No.: +212 661208038

RWANDA

Phillips Pharmaceuticals (Rwanda)
Ltd
GF 68, Kigali Modern City Market
Commercial Street
Kigali, Rwanda

TANZÁNIE

Phillips Pharmaceuticals (Tanzania)
Ltd
Vingunguti Industrial Area
P.O. Box 737
Dae Es Salam, Tanzania
Tel. No.: + 255 782637336

ZIMBABWE

Healthyard Laboratories Pvt Ltd
Cooksey House
2 Simon Mazorodze Rd., Southerton
Harare AZ1095
Zimbabwe
Tel. No.: 263778401114

MAURICIUS

Health Focus Ltd.
33 St. Clement Street
Curepipe 74208
Mauritius
Tel.No.: +230 674 1000

NIGÉRIE

Phillips Pharmaceuticals
(Nigeria) Ltd
122-132 Afprint Industrial Estate
Apapa Oshodi Expressway
Iyana-Isolo, Nigeria
Tel.No.: +234 8056292422

JIŽNÍ AFRIKA

Obsidian Health Ltd
Cosmo Business Park
Malibongwe Drive
Randburg, South Africa 2188
Tel. No.: +27 87 3535600

UGANDA

Phillips Pharmaceuticals
(Uganda) Ltd
N-S Maghavani Complex
5th Street, Industrial Area
Bugolobi
Kampala, Uganda
Tel.No.: 256 772 518 681

ASIE/TICHOMOŘÍ (S VÝJIMKOU ČÍNY)**AUSTRÁLIE**

Abbott Australia
299 Lane Cove Road
Macquarie Park
NSW 2113
Australia
Tel: 61 2 9857 1111

AUSTRÁLIE

CMM Technology
Unit 12, 3 King Edward Road
Osbourne Park
Western Australia 6017
Tel: +61 1300 79 70 30

BANGLADÉŠ

Unimed Limited
Rangs Nasim Square (7th Floor),
House 46
Sheikh Kamal Saroni, Road No. 16
Dhanmondi
Dhaka 1209 Bangladesh
Tel: +8802 9128192-3

BRUNEJ

Transmedic Pte Ltd.
5, Jalan Kilang Barat
9th Floor Petro Centre
Singapore 159349
Tel: +65 6 7371 945

**COOKOVY OSTROVY, FIDŽI,
FR. POLYNÉSIE, SAMOA,
TONGA A TOKELAU**

Abbott New Zealand
Building D, 4 Pacific Rise
Mt. Wellington Auckland 1060
New Zealand
Tel: 64 9 573 6030

HONGKONG

BioAsia Diagnostics Company Ltd
Unit 1-3, 20F
CRE Centre, 889 Cheung
Sha Wan Road, Cheung Sha Wan
Kowloon Hong Kong
Tel: +852 27870906

INDIE

Sandor Medicaids Private Ltd.
8-2-326/5, Road No. 3
Banjara Hills
Hyderabad, India 500 034
Tel: +91 40 233 570 48

INDONÉSIE

Transmedic Pte Ltd.
5, Jalan Kilang Barat
9th Floor Petro Centre
Singapore 159349
Tel: +65 6 7371 945

JAPONSKO

Japan Medical Link Co., LTD
8-1 marunouchi, matsumoto-shi
nagano-ken, JP 390-0873
Tel: +81 263 38 0411

JAPONSKO

Konishi Medical Instruments Co., Ltd
2-1-5awajimachi, chuo-ku
osaka, JP 541-0047
Tel: +81 6 6941 1363

JAPONSKO

Masuda Medical Instruments Co., LTD
50 takedawarayacho, hushimi-ku kyoto,
JP 612-8443
Tel: +81 75 623 7111

JAPONSKO

Nakakita Yakuhin Corporation
3-5-15 marunouchi
naka-ku
aichi-ken, JP 460-8515
Tel: +81 52 971 3681

JAPONSKO

Takeyama Co., LTD
16-1-5 kita6jonishi
chuo-ku
hokkaido, JP 060-0006
Tel: +81 11-0611-0100

JAPONSKO

Abbott Japan
3-5-27, Mita, Minato-ku
Chiba 270-2214
Japan
Tel: 65 6914-8351

INDIE

Crescent Organics Pvt. Ltd.
WINDSOR, 2nd Floor
Cst Road, Kalina, SantaCruz (E)
Mumbai 400098

JAPONSKO

Hokuyaku Inc.
16-1-5 kita6jonishi, chuo-ku
Hokkaido JP 060-0006
Tel: +8 11611-0989

JAPONSKO

Kawanishi Corporation
1-3-9 ima, kita-ku
okayama-ken, JP 700-8577
Tel: +81 86 241 9242

JAPONSKO

Kyowa Medical Corporation
156-2ikededa, suruga-ku
shizuoka-ken, JP 422-8005
Tel: +81 45 655 6600

JAPONSKO

Mitas, Inc.
4-901 tonyacho
hukui-shi
hukui-ken, JP 918-8556
Tel: +81 776 28 2888

JAPONSKO

Sanshodo Co., LTD
68kamitobaomonocho
minami-ku
kyoto, JP 601-8533
Tel: +81 75 681 5131

JAPONSKO

Tomiki Medical Instruments, Co., Ltd
2-46 tonyacho
kanazawa-oshi
Ishikawa-kens, JP 920-8539
Tel: +81 76 237 5555

**KIRIBATI, MIKRONÉSIE,
NOVÁ KALEDONIE A PAPUA
NOVÁ GUINEA**

Abbott Australia
299 Lane Cove Road
Macquarie Park
NSW 2113 Australia
Tel: 61 2 9857 1111

INDIE

Genpharmasec Limited
104/105, 1st Floor
Gundecha Indust. Comp. Premise
Akurli Road, Kandivali East
Mumbai II 400 India

JAPONSKO

Iwabuchi Pharmaceutical Co., Ltd
1-5 Takanodai
Yotsukaido-Shi (Chiba) JP
284-0033
Tel: +81 43 310 6630

JAPONSKO

Kishiya Inc
1-14-21 matsushima
higashi-ku
hukuoka-ken, JP 812-0062
Tel: +81 92611 8000

JAPONSKO

Maruki Medical Systems Inc.
3-20-7 nishinakata, taihaku-ku
miyagi-ken, JP 981-1105
Tel: +81 22 242 6001

JAPONSKO

Musashi Engineering Co., LTD
sekitori60, uedacho
Aichi, JP 441-8134
Tel: +81 532 25 1231

JAPONSKO

Shibataintech
2-11-3 oroshimachi
wakabayashi-ku
miyagi-ken, JP 984-0015
Tel: +81 22 236 2311

JAPONSKO

**Yamashita Medical Instruments
Co., LTD**
hakataza and nishigin building
10
2-1 shimokawabatacho
fukuokaken, JP 812-0027
Tel: +81 956 25 2112

KYRGYZSTÁN

Meditek Ltd.
Ryskulova Str. 12
Bishkek, Kyrgyzstan 720001
Tel.: 00996 312 93 50 05

MALAJŠIE

Transmedic Pte Ltd.
5, Jalan Kilang Barat
9th Floor Petro Centre
Singapore 159349
Tel: +65 6 7371 945

NEPÁL

Synergy Medisales Pvt. Ltd
#205, Bishal Nagar, Bishal Basti kha
Kathmandu, B
977 Nepal
Tel.: +977-01-4425956

PÁKISTÁN

Hospicare Systems
Mezzanine Floor. Rabbiya Garden Block 3
Karachi
74800, Pakistan
Tel. No: +92 2134921913

SINGAPUR

Transmedic Pte Ltd.
5, Jalan Kilang Barat
9th Floor Petro Centre
Singapore 159349
Tel: +65 6 7371 945

SRÍ LANKA

Hemas Surgicals & Diagnostics Pvt. Ltd
No. 75 Braybrook Place
Colombo, Sri Lanka
Tel: 94114766680

THAJSKO

Connect Diagnostics Co., Ltd
1/57-58 Venice Di Iris
Soi Watcharaphol 2/7
Bangkok 10220 Thailand
Tel: 6623470109

MALEDIVY

Medtech Maldives Pvt Ltd
Ma. Fas Eri 3rd Floor
Ameenee Magu Male 20205
Male 20205 Maldives
Tel: +960 3308663

NOVÁ KALEDONIE

Medi-Services S.A.R.L
8, rue Reverce
Noumea
New Caledonia
Tel: 687272000

FILIPÍNY

Euromed Laboratories Phil., Inc.
Sabutan Paliparan Road
Brgy. Sabutan
Silang. Cavite 4118 Philippines
Tel.: 63 (02) 85240091 98

**ŠALAMOUNOVY OSTROVY,
TUVALU, VANUATU A YAP**

Abbott Australia
299 Lane Cove Road
Macquarie Park
NSW 2113
Australia
Tel: 61 2 9857 1111

TAIWAN

Ditech Enterprise Co., Ltd.
9F, No. 168, Sec. 1
Zhongshan Rd
New Taipei City 234 – TA - 234
Taiwan
Tel: +886 2 8923 2236

UZBEKISTÁN

Albatros World
Buyuk Ipak Yoli street, 49
Mirzo Ulugbek district
Tashkent, Uzbekistan 10005 4
Tel: +998 97 744 78 63

MYANMAR (BARMA)

**Advanced Diagnostic Products
Trading Co., Ltd**
No. 34, Thate Pan Street
Ahlone Township
Yangon, Myanmar
Tel.: +951 229587

NOVÝ ZÉLAND

Abbott New Zealand
Building D, 4 Pacific Rise
Mt. Wellington
Auckland 1060
New Zealand
Tel: 64 9 573 6030

FILIPÍNY

**Medical Trends & Technologies,
Inc. (MEDT2EK)**
#200J Elizalde Street, Phase V
BF Homes
Paranaque City 1700
Philippines
Tel: 6388069767

JIŽNÍ KOREA

POC Korea Ltd.
Ace Gasan Tower, 121 Digital-ro
Suite 1501
Gumchon-gu, Seoul
08505 South Korea
Tel: +82 2 2058 0971

THAJSKO

Transmedic Pte Ltd.
5, Jalan Kilang Barat
9th Floor Petro Centre
Singapore 159349
Tel: +65 6 7371 945

VIETNAM

Transmedic Pte Ltd.
5, Jalan Kilang Barat
9th Floor Petro Centre
Singapore 159349
Tel: +65 6 7371 945

ČÍNA

BIOTEC Biotechnology Limited Co.
Room 1013, Wu Yang Xin Chen Plaza
No. 111-115, Si You Xing Road
Yue Xiu District, Guangzhou
People's Republic of China
Tel. No: 86-020-28865008

**Beijing Huiwen Yuanmei
Technology & Trading Co., Ltd**
Room C100, Area C, No. 14 Building
No. 2 Kechuang Dongwu Street
Beijing, China 100111
Tel. No: 010-57621837

**Guangdong Biochem
Healthcare Investment Co., Ltd**
Room 903-904, Building A, No. 263
Shuangcheng International Plaza
Guangzhou, China
Tel. No: 86-020-38372808

**Chongqing Chenjie Medical
Equipment Co. Ltd**
Room 4-6, Building 4
No. 101 Cuibai Rd
Dadukou District
Chongqing, 400080 China
Tel.No: 86-023-68153822

**Sinopharm Group Hebei
Medical Equipment Co., Ltd.**
No. 309 Zhonghua North Street
Xinhua District
Shijiazhuang 050050 China
Tel. No: 0311-67302513

**Sinopharm Medical Supply
Chain Service (AnHui) Co., Ltd**
No. 15, Red Maple Road
Hi-Tech Zone
HeFei
230088 China
Tel. No: 86-551-65310478

**Sinopharm Group Xinjiang Medical
Equipment Co., Ltd**
16th Floor, 1st Building
199 North Liyushan Road
Urumai 830054 China
Tel. No: 86-991-3197595

**Xuzhou Ruida Medical Equipment
Ltd**
Room 306, No. 25, Jinqiao Road
Jinshan Bridge Economic
Development Zone
Xuzhou 221000 China
Tel. No: 86-18761439568

**Shenzhen Zichang Technology
Co., Ltd.**
Room 603, 6 Building, Tongfuyu
industrial city
Tanglang, Xili Street, Nanshan
District
Shenzhen CH
Tel. No: 18124785081

**Beijing Bao Ze Kang
Pharmaceutical Co., Ltd.**
B512/513 5F, 6-1 building
28 red lotus road
Xicheng District
Beijing CH
Tel. No: 010-62358509

**Chengdu Life Medical Equipment Co.
Ltd.**
Room 2-5, 13th Floor, Building No.2
799 Jinfu Rd
Jinniu District, Chengdu city
Sichuan Province
People's Republic of China
Tel. No: 028-65037108

**Realcan Phamaceutical Group
Shanghai Co., Ltd.**
Room 1501, 388 Fenglin Road
Xuhui District
Shanghai
Shanghai CH 200032
People's Republic of China
Tel. No: 86-18153573207

**Wuhan Zhongji Pathology Diagnosis
Center Co., Ltd.**
Room 1, 4th Floor, Building B12
No. 818 High-tech Street
EastLake Development Zone
Wuhan, China
Tel. No: 15071168946

**Shanghai Jointown Medical Devices
Supply Chain CO.,LTD**
Room 206, 2/F, Building 7
No. 285 fengguan Road
Jinshan District
Shanghai, CH
Tel. No: 15207181724

**Guangdong heyidukang Medical
Technology Co., Ltd.**
Room 217, building 7, Jiangmen
Wanda Plaza, Pengjiang district,
Jiangmen
China 529000
Tel. No: 15807502929

**Sinochem Beijing Medical Technology
Co., Ltd**
17 /F, Jincheng Building
Longpan Middle Road
Qinhuai District, Nanjing, China

EVROPA

RAKOUSKO

Abbott Austria
Perfektastr., 84A
Wien, Austria
A 1230
Tel. No.: +43 1 89 122 0

CHORVATSKO A MALTA

Mark Medical d.o.o.
Budmanijeva 5
Zagreb, 10000, Croatia
Tel. No.: +385 1 6065 444

FINSKO

Abbott Oy (Abbott Finland)
Linnoitustie 4
Espoo, Finland 02600
Tel. No.: +358 9 751 8418

NĚMECKO

Abbott GmbH (Abbott Germany)
Max-Planck-Ring 2-3
65205 Wiesbaden – Delkenheim
Germany
Tel.No.: +49-6122-58-0

IRSKO

Abbott Laboratories, Ireland Limited
Block B, Liffey Valley Office Campus
Dublin 22
D22 XOY3
Tel. No.: +353 1 469 1560

ITÁLIE

Tobar Service SRL
Via Veronica Gambarà 23
Rome, Italy
Tel. No.: +39 06 89161897

POLSKO

Diag-med
ul.Stanislawa 50
Prusków
05-800 Poland
Tel.No.: +48 22 8389723

ÁZERBÁJDŽÁN

Albatros Healthcare
Y. Haseynov 6
Narimanov
Baku, Azerbaijan AZ1021
Tel.No.: +994 125648635

ČESKÁ REPUBLIKA

Coro Medical s.r.o.
Biskupsky dvur 2095/8
Prague, Czech Republic 110 00
Tel.No.: 42603229000

FRANCIE

Abbott France S.A
40/48 rue d'Arcueil
94593 Rungis Cedex, France
Tel. No.: +33 1 4560 2500

NĚMECKO

WS Laborbetriebsgesellschaft GmbH
Langendembach 72A
Langenoria GE 07381
Germany
Tel.No.: +49 3647 – 414378

ITÁLIE, VATIKÁN, MALTA A SAN MARINO

Abbott SRL (Italy)
Via Ribotta 9
Rome, Italy 00144
Tel. No.: +39 06 52 9911

NIZOZEMSKO

Abbott B.V. (Abbott The Netherlands)
Wegalaan 9
2132 Hofddorp
Netherlands
Tel.No.: +31 88 82 22 500

PORTUGALSKO A ŠPANĚLSKO

Magnamed, Lda
Rua Prof Francisco Gentil n° 22F
2620-097 Póvoa de Santo Adrião
Portugal
Tel.No.: +351 21 938 32 40

BELGIE

A LUCEMBURSKO
Abbott sa/nv
(Abbott Belgium)
Avenue Einstein 14
1300 Wavre, Belgium
Tel.No.: +32 10 47 53 11

DÁNSKO, GRÓNSKO, ISLAND A FAERSKÉ OSTROVY

Abbott Laboratories A/S
(Abbott Denmark)
Emdrupvej 28C
DK-2100
Copenhagen, Denmark
Tel. No.: +45 39 77 00 00

GRUZIE

Advanced Medical Technologies & Services AMTS
Saburtalo Str. 7, Flat 13
Tbilisi 0159
Georgian Republic
Tel.No.: +995 32 532 032/036

ŘECKO, KYPR, ALBÁNIE A KOSOVO

Pegasus Health and Sciences S.A.
Thessalonikis 18
Chalandri
Athens, GR 15234
Tel.No.: +30 210 6139496

ITÁLIE

Burke & Burke
Via A Einstein, 32
Assago (MI)
20090 Italy
Tel. No.: +39 02 4571 3663

NORSKO

Abbott Norge as (Abbott Norway)
Martin Linges vei 25
Postboks 1 Fornebu N-1330
Norway
Tel. No.: +47 81 55 99 20

RUMUNSKO

MD&D (SC Medical Devices & Diagnostics SRL)
111C, Drumul Vales Furcii St.
Sector 6
Bucharest 061985
Romania
Tel.No.: 40311054280

RUMUNSKO
Ducos Trading Ltd
Johann Strauss, No. 2A
Etaj 3, Camera 5A
Bucuresti 020312
Romania
Tel. No: +40 314 381 094

RUSKO
Eco-med-s M LLC
Samokatnaya str., 2A
Bld. 1 p.105, r. 3, of. 2
Moscow, Russia 111033
Tel. No: +7 (495) 748-43-50

**ŠPANĚLSKO, PORTUGALSKO,
GIBRALTAR, ANDORRA
A KANÁRSKÉ OSTROVY**
Abbott laboratories SRL
(Abbott Spain)
Parque Empresarial Via Norte
c/ Quintanavides
17 Madrid 28050 Spain
Tel. No.: +34 91 337 3400

**ŠVÝCARSKO
A LICHTENŠTEJNSKO**
Axonlab AG - Switzerland
Taefernstrasse 15
CH-5405 Baden-Daetwil
Switzerland
Tel.No.: +41 56 484 8080

SPOJENÉ KRÁLOVSTVÍ
Abbott Laboratories Limited
(Abbott UK)
Abbott House
Vanwall Business Park
Maidenhead
SL6 4XF, United Kingdom
Tel.: 44 16 28 784 041

RUSKO
Abbott Laboratories Ltd (Abbott
Russia)
Leningradskoe shosse 16A
Bld 1, Moscow
Russia 125171

RUSKO
UDS (Updated Diagnostic Systems)
3 Bogorodsky Vall
Moscow, Russia 107076

ŠPANĚLSKO
Techmédica
C/ Luis Doreste Silva, 62 - 1°
Las Palmas
35004 Spain
Tel. No.: +34 928 242381

**ŠVÝCARSKO
A LICHTENŠTEJNSKO**
Abbott AG (Abbott Switzerland)
Neuhofstrasse 23
CH-6341 Baar
Switzerland
Tel. No: +41 41 768 44 44

RUSKO
Servis Instrument LLC
Bld 1, Room 18
Ryabinovaya Str. 38
Moscow 121471
Russia
Tel. No: +7 495 628 7845

**SLOVINSKO, BOSNA
A HERCEGOVINA**
EUROMED, d.o.o.
Podpeska cesta 14
Brezovica pri Ljubljani
Slovenia, SI-1351
Tel. No: +386 59 338 202
+386 40 451 115

ŠVÉDSKO
Abbott Scandinavia AB (Abbott
Sweden)
Hemvärnsgatan 9
Solna SE17129
Sweden
Tel. No.: +46 (0)8 546 567 18
E.post: Service@abbott.se

TURECKO
Gazi Kimya Tip Teknolojileri
San. Ve Tic. Anonim Sirketi
Serifali Mah. Beyan Sokak
Akalin Plaza, No. 3
Umraniye, Istanbul Turkey
Tel. No.: 0090 532 462 55 69

LATINSKÁ AMERIKA

ANGUILLA

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel. No: 407 475 1168

ARUBA

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

BELIZE

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

BONAIRE

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

BRAZÍLIE

Medcare Produtos
E Equipamentos Medico Hospitalares LTDA
Al Tres de Outubro, 795
Sarandi-Porto Alegre
RS, Porto Alegre
91130-470, Brazil
Tel. No: +51-984466491

CHILE

Alatheia Medical SpA
Avenida del Valle Norte 945
Oficina 5610
Huechuraba, Santiago
Chile
Tel.: +562 343 5122

KOLUMBIE

Comprolab
CALLE 106 #54-63
Bogota
Colombia

DOMINIKA

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

ANTIGUA A BARBUDA

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

BAHAMY

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

BERMUDY

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

BRAZÍLIE

Tecno4 Produtos Hospitalares Ltda.
Rua Backer
89 Cambuci
Sao Paulo
Brazil 01541-000
Tel.: +1 55 11 95557-6345

BRITSKÉ PANENSKÉ OSTROVY

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

KOLUMBIE

Abbott Laboratories de Colombia S.A.S
Calle 100 #9A-45
Piso 14 Bogota
Colombia
Tel.: (57 1) 628 5600

KOSTARIKA

Tri Dm
Calles 16-18 Ave 8
No. 1628
San Jose, CR 10103
Tel.: +506 2257 7676

DOMINIKÁNSKÁ REPUBLIKA

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

ARGENTINA

Drogueria Artigas S.A.
Av. Jose Luis Chorroarin, 1079
Buenos Aires Capital Federal
Argentina C1427CXH
Tel.: +64 3 338 0999

BARBADOS

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

BOLÍVIE

Sanimed S.R.L.
Calle Cordero No 130 Zona
San Jorge, LaPaz, Bolivia
Tel.: 591-2 2431294

BRAZÍLIE

PMH Produtos Medicos
Hospitalares LTDA
SIA Trecho
17 RUA 8 LOTE 170
Brasilia, DF Brazil 71200-222
Tel.: +55 61 81312112

KAJMANSKÉ OSTROVY

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

KOLUMBIE

Arrow Medical SAS
Calle 23# 43 A-100
Medellin, Antioquia
Colombia
Tel.: +574 356 1111

CURAÇÃO

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

EKVÁDOR

M. R. U.
Av. Eloy Alfaro N29-235
Entre Italia y Alemania P
Quito, Ecuador
Tel.: +593-2 3801 533

SALVADOR**Servicios Quirúrgicos**

Km. 11 Carretera al Puerto de la Libertad
 Antigua Cuascatlan
 La Libertad, El Salvador
 Tel.: +503 2241-6200

SALVADOR**Interserv SA de CV - El Salvador**

67 Avenida Sur
 Final Pasaje "A", No. 4-B
 Colonia Roma
 El Salvador Cp 1101
 Tel: +503 2223-5510

GRENADA**American Hospital Supply (AHS)**

1060 Maitland Center Commons
 Maitland, FL USA 32751
 Tel.: 407 475 1168

GUADELOUPE, GUYANA**A MARTINIK****Abbott France S.A**

40/48 rue d'Arcueil
 94593 Rungis Cedex, France
 Tel.No.: +33 1 4560 2500

GUATEMALA**Promed S.A.**

Avenida Ferrocarril 19-97
 Empresarial El Cortijo Interior 202
 Zona 12, Guatemala 10001
 Tel.: +502 2223 86 00

GUYANA**American Hospital Supply (AHS)**

1060 Maitland Center Commons
 Maitland, FL USA 32751
 Tel.: 407 475 1168

HAITI**American Hospital Supply (AHS)**

1060 Maitland Center Commons
 Maitland, FL USA 32751
 Tel.: 407 475 1168

HONDURAS**ProdyLab S de R.L.**

Col. Santa Ana, 11 y 12
 Avenida 12 Calle Bloque No. 117,
 N.O Sand Pedro Sula, Honduras
 Tel.: +504 2550-3091

JAMAICA**American Hospital Supply (AHS)**

1060 Maitland Center Commons
 Maitland, FL USA 32751
 Tel.: 407 475 1168

MEXIKO**Abbott Laboratories de Mexico SA de CV**

Calzada de Tlalpan, 3092
 Colonia Ex Hacienda Coapa
 D.F. 04980 Mexico
 Tel.: (5255) 58097500

MEXIKO**Abalat S.A. de C.V.**

Abasolo 78
 Pedregal de Sta Ursula
 Coyoacán, Mexico City MX 04650
 Tel.: 80001500

MEXIKO**Casa Plarre S.A. de C.V.**

Av. Cuauhtemoc 220, interior
 201 Colonia Doctores
 Alcaldía Cuauhtemoc
 Ciudad de Mexico

MEXIKO**Centrum Promotora Internacional S.A. de C.V.**

Medellin 324
 Roma Sur, Cuauhtémoc
 Mexico City, MX 06760
 Tel.: +52 5552652500

MEXIKO**Comercializadora de Reactivos**

Ave. Eugenio Garza Sada N°.6336
 Col. Ciudad Satélite
 Monterrey, MX 64960
 Tel.: 800 7107 566

MEXIKO**Diagmex S.A. de C.V.**

Calle San Martin de Porres 3777
 y 3769
 Jardín de San Ignacio
 Zapopan, Jalisco MX 45040
 Tel.: 01 8008770007

MEXIKO**Diagnóstica Internacional**

Circunvalación Sur 156
 Las Fuentes
 Zapopan, Jalisco MX 45070
 Tel.: +52 3337771940

MEXIKO**GRUPO ICTZE SA DE CV**

Callejon SN 172 Parcela 29
 Col. Playa de vacas
 Medellin, Veracruz MX 94274
 Tel.: 2299357143

MEXIKO**Grupo Biomédico Empresarial**

Calle Tlacoquemecatli 21
 Tlacoquemecatli del Valle
 Benito Juarez
 Mexico City MX 03200

MEXIKO**Impromed S.A. de C.V.**

Perpetua 30, Int. 101
 Col san José Insurgentes
 Alcaldía Benito Juarez
 Mexico City MX 03900
 Tel.No: +52 5553228870

MEXIKO**Instrumentos y Equipos Falcón S.A de C.V**

Calle José María de Teresa 188
 Campestre – Álvaro Obregón
 Mexico City MX 01040
 Tel.No: 52 55 17190600

MEXIKO**LAVKEM SAPI de CV**

Paseo de los Tamarindos, 90
 Torre 1, Piso 32
 Ciudad de Mexico

MEXIKO

Ruvel S.A. de C.V.
Av Lago de Guadalupe KM 1.5
San José el Jaral
Cd López Mateos, MX 54927
Tel.No: +52 5553701047

PANAMA A GUATEMALA

Promed S.A.
Pque. Industrial Costa del Este,
C/ 2da Edi.Premed
Panama
Tel.: +507 303-3100

PORTORIKO

Cardinal Health Puerto Rico
Centro Internacional de Distribution
Carretera 869 KM 4.2
Guaynabo 00695 Puerto Rico
Tel.: 787 625 4100

SVATÝ MARTIN

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

TORTOLA

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

URUGUAY

Bioerix S.A.
Simon Bolivar 1472
Montevideo Uruguay
Tel.: 59827081624

MEXIKO

**SERVICIOS INTEGRALES E
INGENIERIA HENLIS**
Bosque de las Palmas
Naucalpan de Juárez, MX 52787
Tel.No: 52722 216 1905

PARAGUAY

Index S.A.C.I.
Boqueron No. 676
Postal Code 1414
Asuncion, Paraguay
Tel.: 595 21 214 213

SVATÝ KRYŠTOF A NEVIS

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

**SVATÝ VINCENC
A GRENADINY**

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

TRINIDAD A TOBAGO

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

MONTSERRAT

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

PERU

Biomedical Systems, S.A.
Av. Rafael Escardo
No. 823, San Miguel
Lima 32 , Peru
Tel: +511 241 3843

SVATÁ LUCIE

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

SURINAM

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

TURKS A CAICOS

American Hospital Supply (AHS)
1060 Maitland Center Commons
Maitland, FL USA 32751
Tel.: 407 475 1168

BLÍZKÝ VÝCHOD

ÍRÁN

Paya Zist Arayeh
No. 13, 10th Alley
Zarafshan St., Farahzadi St.
Shahrak-e-Gharb, Tehran
Iran
Tel: 989122836214

KRÁLOVSTVÍ BAHRAJN

Gulf Corporation for Technology
Bldg 2038, Road: 4156, Block: 341
Manama/Al Juffair
Kingdom of Bahrain
Tel: 97317240597

OMÁN

Waleed Pharmacy & Stores LLC
Street No. 40, Building No. 176
North Al Ghoubra, Ministry Road
Ghoubra
Muscat 100 Oman
Tel: 96899338480

SAÚDSKÁ ARÁBIE

**Medical Supplies & Services Co. Ltd
(MEDISERV)**
Mediserv Building
Alahassa Street
Riyadh, Saudi Arabia
Tel.No.: +966 1 478 0555

IZRAEL

Rhenium Ltd.
20 Hasatat st PO Box 180
Modi'in 71711 Israel
Tel: +972-8-9558888

KUVAJT

Bader Sultan & Bros. W.L.L.
Block B 67, Shuwaikh, Future Zone
13009 Safat
Kuwait
Tel: +965 2461 0280

PALESTINA

**Medical Supplies and
Services (MSS)**
Betunia, Unipal building
Ramallah Palestine
Tel.No.: 00972 2 2959372

SPOJENÉ ARABSKÉ EMIRÁTY

Gulf & World Traders
P.O. Box 5527, Al Garhoud Area
Dubai
Tel.No.: +971 4 2821717

JORDÁNSKO

Smart Care Medical Co.
Wasfi El tall street
al Fannar complex
Amman, Jordan
Tel: 1 55 11 95557-6345

LIBANON

Union Pharmaceutique d Orient
Museum Street Badaro
Beirut 11-0709
Lebanon
Tel: +96 113 87815

KATAR

**Khalid Scientific
Company W.L.L.**
Airport St., Bldg. No. 402
Area No. 47
Doha
Qatar 4349
Tel: 974 432 5198

JEMEN

Griffin Ltd.
Near Linton R/A 13
Opposite Mineral Circles
Bearing Free St.
Jebel Ali, 732 UAE
Tel. No.: +967 1 441 907