ID NOW™
INSTRUMENT (NAT-000)
SOFTWARE RELEASE 7.3.0.13

ID NOW™

INSTRUMENT (NAT-000)

SOFTWARE RELEASE 7.3.0.13

Software Release 7.3.0.13	1
Verze softwaru 7.3.0.13	3
Softwareversion 7.3.0.13	5
Softwareversion 7.3.0.13	7
Έκδοση λογισμικού 7.3.0.13	9
Versión de software 7.3.0.13	11
Ohjelmistoversio 7.3.0.13	13
Version du logiciel 7.3.0.13	15
Izdanje softvera 7.3.0.13	17
Szoftverkiadás 7.3.0.13	19
Rilis Perangkat Lunak 7.3.0.13	21

Versione software 7.3.0.13	23
소프트웨어 릴리스 7.3.0.13	25
7.3.0.13 programinés įrangos laida	27
Programmatūras laidziens 7.3.0.13	29
Softwareversie 7.3.0.13	31
Programvareversjon 7.3.0.13	33
Versão de software 7.3.0.13	35
Versiune de software 7.3.0.13	37
Выпуск программного обеспечения версии 7.3.0.13	39
Vydanie softvéru 7.3.0.13	41
Izdanje softvera 7.3.0.13	43
Programvaruversion 7.3.0.13	45
Yazılım Sürümü 7.3.0.13	47
Bản phát hành Phần mềm 7.3.0.13	49
軟體版本 7.3.0.13	51

Software Release 7.3.0.13

Abbott Diagnostics Scarborough, Inc. has released an updated version of software for the ID NOW™ Instrument, part number NAT-000. This release includes the following updates and enhancements:

Allows the user to run the following:

- ID NOW™ COVID-19 2.0 (193-000)
- ID NOW™ Influenza A & B 2 (427-000)
- ID NOW™ Strep A 2 (734-000)
- ID NOW™ RSV (435-000)

Software enhancements for:

Cybersecurity & Workflow Updates:

- Improvements to instrument data security through encryption of data at rest.
- Workflow enhancements for test result export and Admin-controlled deletion, providing greater control over locally stored test results.

The version of software currently installed on an ID NOW Instrument can be found under the Preferences > About > Info screen.


IMPORTANT: The Software Upgrade Kit Contains the following:

- USB Drive containing:
 1. ID NOW Instrument Software Update, version 7.3.0.13
 2. ID NOW Instrument User Manual, part number INNAT000
 3. ID NOW Instrument Quick Start Guide, part number INNAT009
- ID NOW Instrument OUS Software Release Technical Bulletin, part number TB000021
- Abbott Software Upgrade Confirmation Form, part number INNAT300

The ID NOW Instrument can be updated using a Software Upgrade kit containing a USB drive. The user inserts the USB drive and powers up the instrument to apply the update. See detailed procedure instructions below.

USB Software Update Cautions:

- ⚠ **Connectivity:** If the ID NOW is connected to a LIS/EMR, ensure the data management system is on the most up to date version before completing the ID NOW software update.
- ⚠ **Stored Data:** The software update does not delete User, Lot, or Patient Result data. However, it is best practice to export any needed data before proceeding as there is potential for corruption.
- ⚠ **USB Drive:** DO NOT REMOVE USB drive during software update. USB drive may only be removed from the ID NOW as per the update sequence below. Only one USB drive may be connected during the software update process.
- ⚠ **External Quality Control:** External quality controls are not required post software update. External quality controls are only required on each ID NOW Instrument for any newly added assays.
- ⚠ **ID NOW Settings:** Confirm all previously configured instrument settings post software update.

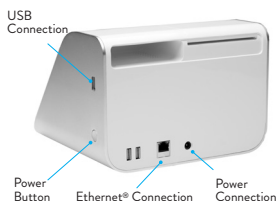Procedure:

Step 1: Turn the instrument OFF. Press and hold the Power Button for 3-5 seconds.

Step 2: Attach USB Drive (provided) to the instrument via the **USB Connection**. Wait at least 3 seconds before proceeding to Step 3.

Step 3: Turn the instrument ON by pressing and holding the Power Button for 3-5 seconds until the display on the screen is shown and the white LED on the front begins to blink. A message, "Verifying Update", will appear about 45 seconds after turning on the ID NOW Instrument.

Step 4: "Touch to Install" option will appear on the screen approximately 1.5 minutes after turning the instrument on.

Touch the ID NOW display screen to begin the install.

Do not remove the USB Drive.

Note: Do not log in before completing Step 5.

Step 5: After the update has been installed, remove the power cord from the back of the instrument. Remove the USB Drive from the side of the instrument. Wait 10 seconds. Reconnect the power cord to the instrument. The white LED on the front will blink a few times. Turn on the instrument. The software update is complete. The USB can be used again for additional instruments.

Note: Verify the update was successful by selecting "Preferences" > "About" > "Info". The software version will be displayed. More info is in the ID NOW Instrument User Manual, Section 5.5.

External quality controls are not required post software update. External quality controls are only required on each ID NOW Instrument for any newly added assays.

If unexpected results are obtained or you are unable to complete the software update, please contact Abbott Technical Support.

Should you have any further questions, please contact Abbott Technical Support:

US	+ 1 855 731 2288	ts.scr@abbott.com
Africa, Russia, CIS	+ 44 161 483 9032	EME.TechSupport@abbott.com
Asia Pacific	+ 61 7 3363 7711	AP.TechSupport@abbott.com
Canada	+ 1 800 818 8335	CANproductsupport@abbott.com
Europe & Middle East	+ 44 161 483 9032	EME.TechSupport@abbott.com
Latin America	+ 57 601 4824033	LA.TechSupport@abbott.com

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© 2024 Abbott. All rights reserved.
All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners.